REGENhial INTIM Gel

LaviGor
LABORATORIOS

CN 179694.3

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Presentation:
2 tubes of 24 ml
2 applicators

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Vaginal moisturising and repairing gel

Description

Vaginal moisturising and repairing gel based on pure hyaluronic acid (*) at 1%, arginine and glycerol.

Indications

Regenhial Intim Gel relieves the symptoms of vaginal dryness, atrophy and the troubles related to itching, stinging and dyspareunia (painful sexual intercourse), caused during pregnancy, puerperium, menopause or as a consequence of physical or chemical irritation, or associated to the use of oral contraceptives or other medicines or treatments. It moisturises, repairs and maintains the elasticity of vaginal and perivaginal tissues. Besides, it improves lubrications, thus it is also indicated to facilitate sexual intercourse.

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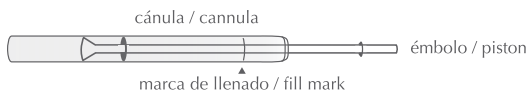
Instructions for use

Wash your hands before and after use.

Regenhial Intim Gel is to be applied inside the vagina with the help of the applicator, previously filled with 2 ml.

It is recommended one application of Regenhial Intim Gel twice or three times a week or can be used daily if needed, preferably before going to bed.

Componentes: 2 tubos y 2 aplicadores (cánula + émbolo).
Components: 2 tubes and 2 applicators (cannula + piston).

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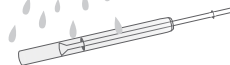
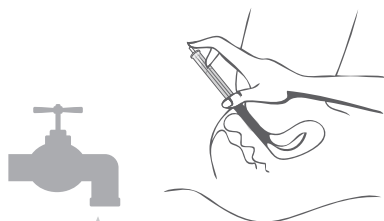
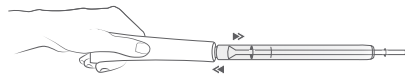
Press the tube and gently fill the piston until the fill mark on the non-screwed end of the cannula is observed. The gel dose corresponds with a little less of half of dispenser capacity and the presence of air inside must be avoided.

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Unscrew the applicator from the tube and introduce it deeply in the vagina, preferably in decubitus position, and empty the whole dose of **Regenhial Intim Gel** by pulling the piston gently. It is recommended to apply a thin layer of **Regenhial Intim Gel** on the end top of the applicator in order to facilitate its introduction into the vagina.

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After each application, clean the applicator with soapy water and dry it properly. It is recommended to use a different applicator for each tube.

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Contraindications

Do not apply in case of confirmed or under suspicion hypersensitivity to any of the components of **Regenhial Intim Gel** (see composition).

Warnings

Do not use in case of any alteration in the packaging. • Do not use the product after the expiry date indicated on the packaging. • Do not ingest. • Avoid contact with eyes. • Keep at room temperature, away from heat sources and avoid direct light. • Keep out of reach and sight of children. • After use, place in the appropriate waste container, according to the country applicable practice.